

WOUNDSEAL®

Powder + Pressure = Instant Scab

Usage Instructions

WoundSeal Pour Pack

Clean wound and apply only when bleeding resumes. WoundSeal Powder needs blood to work.



1. Tear package open where indicated.



2. Completely cover wound with powder.



3. Press for 30 seconds to form the protective scab and stop the bleeding.

WoundSeal Rapid Response

Clean wound before application.



1. Open package where indicated.



2. Pour powder on bleeding wound and apply pressure directly over powder. On large wounds, pour over wound in sections, applying pressure as you go. Pressure is needed to force powder onto wound bed.



3. Hold pressure for up to 1 minute or until scab forms and bleeding stops.

Seek further medical attention as needed. Show product removal directions on package to medical personnel.

WoundSeal + Applicator

For hard-to-reach wounds: Clean wound and apply only when bleeding resumes. WoundSeal Powder needs blood to work. Use enclosed applicator to apply powder.



1. Peel blister and load applicator



2. Apply as much powder as possible, then apply pressure for 30 seconds.

BIOLIFE®

800-722-7559
www.biolife.com